

Complications can be biological (integration loss) or mechanical (component fracture due to excessive load). If no complications occur, the duration of the devices and of the prosthesis as a whole depends on the mechanical resistance of the device with respect to the accumulated fatigue.

Sweden & Martina has subjected PRAMA implants to the required fatigue resistance tests at 5,000,000 cycles, and the implants passed the test.

Fatigue tests are carried out in compliance with the specific normative and the results assessed by performing calculations on the finished elements.

12. EXPIRY DATE
Implants must not be used after the expiry date indicated.

13. LEGAL REFERENCES
The design and production of PRAMA implant fixtures are carried out in conformity with the directives and most up-to-date harmonised normatives regarding the materials used, production processes, sterilisation, information provided and packaging.

14. WASTE DISPOSAL PROCEDURES
Fixture implants, if removed from the oral cavity as a result of a biological or mechanical failure, must be treated as organic waste for their disposal, according to the laws that apply locally.

On the other hand, if the implants are sent to Sweden & Martina with a request for execution of a Surf Test, the protocol given on the website www.sweden-martina.com must be followed.

15. LIABILITY FOR DEFECTIVE PRODUCTS AND TERMS OF WARRANTY
Excellent patient care and attention to their needs are necessary conditions for the success of the implant. It is therefore necessary to select the patient carefully, inform them of the inherent risks and of the duties associated with the treatment, encouraging them to co-operate with the dentist to achieve a good outcome from the treatment. The patient must therefore maintain a good level of oral hygiene - confirmed by means of regular check-ups and control visits - that must be guaranteed and documented, along with the pre- and post-surgical directions and prescriptions.

The instructions provided by Sweden & Martina are available at the moment of treatment and have been accepted by the Dental Practice. These instructions must be observed and applied during all the care phases: from the patient medical history stage to the post-surgery check-ups.

The Warranty covers exclusively defects that are established to be attributed production-related and on submission of the piece identified by item and batch code, within validity period of the Warranty.

The Warranty Conditions are available on the www.sweden-martina.com website.

16. DATE AND VALIDITY OF INSTRUCTIONS FOR USE
These Instructions for Use have validity and effect from the month of July 2021.

Table 01

Device	Packaging	Regulation (EU) 2017/745	Classification Rule	Risk Class
Implant fixtures for dental use, belonging to the Prama implantological system	Sterile, single-use packages. Fixtures come complete with cover screw	Implantable devices intended for long term use (over 30 days)	8	IIB
Cover screws	Sold either complete with appropriate fittings or individually (single-use, sterile package)	Implantable devices intended for long term use (over 30 days)	8	IIB

Table 02

Ø 3.30	Light blue
Ø 3.80	Green
Ø 4.25	Blue
Ø 5.00	Magenta

picture 01




picture 02



picture 03-04



picture 05

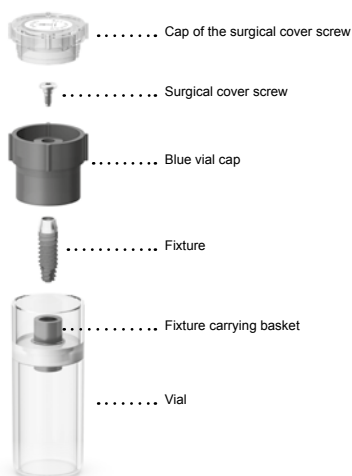

EXPLANATION OF SYMBOLS

	Caution! See instruction for use	✓
	Batch number	✓
	Code	✓
	Manufacturer	✓
	Country of manufacture	✓
	UDI code, Unique Device Identification	✓
	Medical Device	✓
	Consult instruction for use www.sweden-martina.com	✓
	CE marking Where applicable: The identification number of the Notified Body shall follow this symbol.	✓
	American federal law restricts this device to sale by or by order of a professional practitioner	✓
	Do not resterilize	✓
	Disposable product, do not reuse	✓
	Do not use if the packaging is damaged	✓
	Sterilized with ionizing radiation	✓
	Single sterile barrier system with protective packaging inside	✓
	Expiry date after which the product must not be used	✓



Sweden & Martina S.p.A.
Via Veneto, 10 - 35020 Due Carrare (Padova) - Italy
Tel. +39.049.9124300 - Fax + 39.049.9124290
e-mail: info@sweden-martina.com
www.sweden-martina.com