

S.V.G. Smart View Glasses

Quick Guide


1. Plug in the “**Wi-Fi Extender**” device inside your own AC socket and wait 30 seconds.
2. From your own PC: check and connect “**SVG by DTU**” from the Wi-Fi list.
3. Launch 3D Scanning Software and follow your usually steps to begin for scanning.
4. Power on holding “**Power button**” for 5 seconds and release when the led are green.



5. Wear the *Smart Glasses* and wait the **SVG's Home** page as image below.
(Starting around in 30 seconds)




6. Use the “Track Pad” of the “Controller” selecting “Connect” icon as below image:

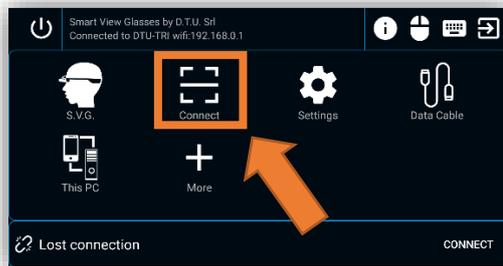

7. “Click” on the “Client” to connect and start to use your 3D Scanning Software (ie: DTU-TRI)

NOTE: If it is not appear any PC, be sure to run the “SVG Receiver” windows application from your own PC and/or the Smart Glasses is connected with the Wi-Fi Extender.

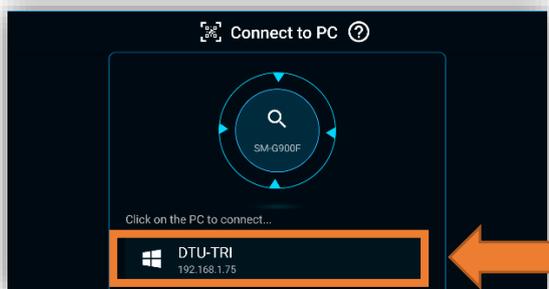

HOW TO:

LOCK/UNLOCK use the “Controller” holding “Switch Key” button for 5 seconds.

POWER OFF hold “Power button” button from the controller for 2 seconds and click on “Power Off” with the “Track Pad”.

